AKVO

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
REH4MAT reserve the right to introduce technical and trading changes in the content of the instruction without warning.

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1. The product has to be used with compliance to guidelines included in this user manual! Before every using, there is necessary to prepare the patient and the device!
2. Before using the product, it is essential to make sure that there are no damages or different faults which could endanger patient's life. Make sure that all parts are operating correctly and are properly mounted. Moreover, make sure that there are no cracks as well as deformities
3. It is forbidden to use platform for different purposes than it is intended to.
4. It is forbidden to make any changes concerning seat's structure due to the fact that it is dangerous and it results in loosing warranty.
5. Before using the product, there is necessary to protect the deckchair or bath chair with included straps with Velcro.

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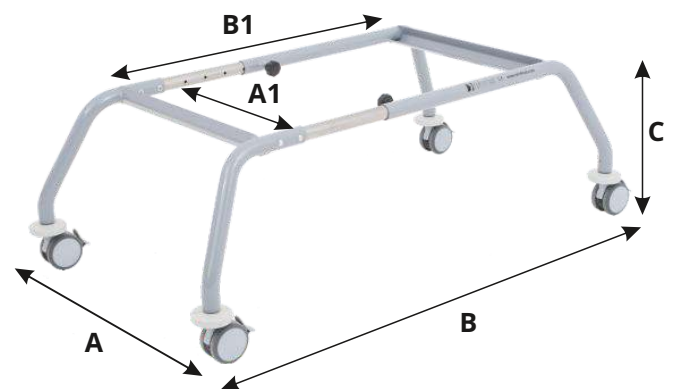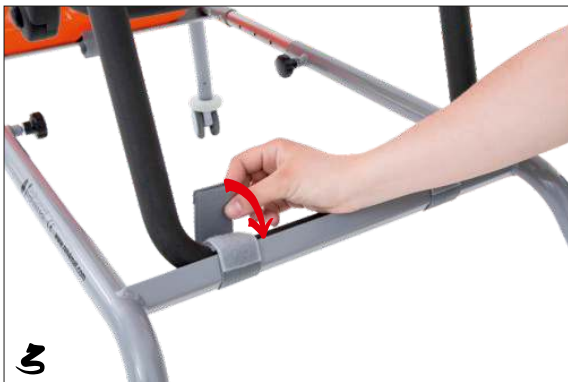
In case of deckchair **AKVOLITO™** assembly, the front strap should go through the special whole in the upholstery

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