

Illustration 1

## Introduction

The users manual contains information about how MeylandDiscover is adjusted, how it is maintained, safety precautions and technical data.

MeylandDiscover is intended as a crawling aid for small children and is only suited for indoor use.

In illustration 1 the different parts of the crawling aid is designated.

## Getting ready for use

MeylandDiscover is supplied fully assembled and must therefore simply be adjusted to fit the child before use.

## Adjustment

The height of the sling support is adjusted on each side with the spring mountings placed on the stanchions. *It is recommended to always set the height equally on each side.* The height is adjusted by drawing out the spring mounting and setting the carry brace to the desired height. (see illustration 2)

MeylandDiscover does not have brakes. All four wheels are smooth rolling and revolving to give the child as much freedom of movement as possible.

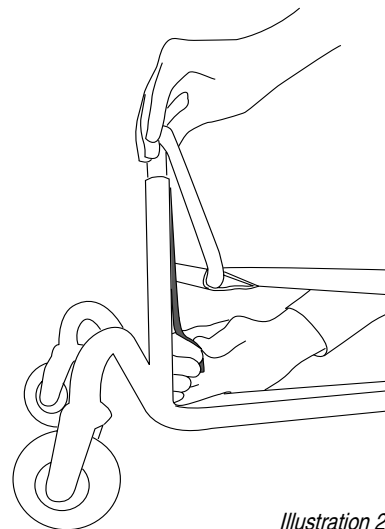


Illustration 2

## Safety precautions

### Warning:

MeylandDiscover must never be used without an adult present and never in places where there is a risk that the child can run over a dangerous edge (stairs or the like).

### Warning:

MeylandDiscover must never be used for sitting or standing on since it can easily roll away under the child.

### Limitations for use:

The maximum allowable weight of the users is 25 kg.

MeylandDiscover should only be used on a plane, horizontal and stable surface.

## Maintenance

All parts of MeylandDiscover can be cleaned with hot water and soap.

It should regularly be checked that bolts, screws and nuts are tightened and if necessary re-tighten them.

## Technical data

	Size 1	Size 2
Total width	52 cm	52 cm
Total depth	31 cm	31 cm
Slingsupport width	28 cm	28 cm
Slingsupport depth	18 cm	24 cm
Height of slingsupport	10 - 21 cm	10 - 21 cm
Weight	4 kg	4 kg
Max load	25 kg	25 kg

The frame is made from powder coated steel pipes and the sling support is made from imitated leather.



User manual **meyland***Discover*  
Size 1 and size 2

**meyland-smith a/s**

Industrivej 27 DK-9830 Taars

Tel: +45 98 96 19 85 Fax: +45 98 96 19 86  
www.meyland-smith.dk info@meyland-smith.dk

**meyland-smith**  
mobility and more