



Etac My-Loo

Raised toilet seat,
with fixed mounting

 **etac**[®]

Next level – A new generation of toilet aids that creates possibilities



My-Loo raised toilet seat with fixed mounting belongs to a whole new generation of smart products from Etac. By setting the highest requirements on everything from safety and comfort to adaptability and ease of installation, we can present a range that offers innovative, unique solutions.

With this new product series, we quite simply take toilet aids to a new level. ‘Design drives development’ has always been our guiding principle, and we are convinced that Etac My-Loo will help users and prescribers to create new possibilities.



Patented arm support solution

Etac My-Loo's innovative arm support feature has an entirely unique height-adjustment mechanism. In addition, the armrests can be conveniently folded up and down with no pinching risk for the user. This patented solution provides optimal individualised support around the toilet.



The height of the armrests is set with a simple manual adjustment. Choose a setting by sliding the orange control into one of the three positions. The steps are clearly marked with 1, 2, 3 on the armrest.

The armrests are mounted from above – perfect for confined spaces. The armrests are marked with L and R - intended for the user's left and right side respectively.

For dismantling, the armrest is folded up and pulled straight upwards to remove it from its bracket.

The armrest can be locked into the desired position using a screw (included).

When only one armrest is used, the opening of the empty bracket is shielded by a cover (included).

Etac My-Loo

With a unique, patented arm support solution, a well-designed seat and quick, simple installation, My-Loo with fixed mounting offers safe support, a high level of comfort and ease of handling. The raised toilet seat is suitable for most toilets and sits securely in place.

Developed to give independence with dignity

My-Loo enables users with reduced mobility or balance problems to manage toilet visits independently and safely. The raised seat and stable armrests make it easier to sit down and stand up from the toilet. An extra-large seat opening and generous hygiene recesses make personal hygiene easier.

Available in five different versions

Etac My-Loo with fixed mounting is available in different designs, with or without armrests and in seat heights of 2, 6 and 10 cm. If necessary, armrests can be added later to a model without armrests.



Etac My-Loo raised toilet seat with fixed mounting.

| | Item no. |
|------------------------------|----------|
| My-Loo with armrests, 20 mm | 80301510 |
| My-Loo with armrests, 60 mm | 80301511 |
| My-Loo with armrests, 100 mm | 80301512 |
| My-Loo, 60 mm | 80301513 |
| My-Loo, 100 mm | 80301514 |

Weight

| | |
|--------------------------------------|------------------------|
| My-Loo with armrests, 20 mm: 3.2 kg | My-Loo, 60 mm: 2.2 kg |
| My-Loo with armrests, 60 mm: 3.4 kg | My-Loo, 100 mm: 2.4 kg |
| My-Loo with armrests, 100 mm: 3.7 kg | |

Maximum user weight

150 kg
Together with accessory Supporting legs, 80301603: 200 kg

Material

Armrest: Polypropylene
Seat/Lid: Polypropylene
Attachment: Stainless steel, Polypropylene, TPE

Design

Myra Industriell Design AB

Maintenance




Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution. The product can be safely cleaned in a cabinet washer-disinfector at 85 °C for 3 minutes.

Features



Adjustable armrests

Using a unique, patented solution, the armrests can quickly and easily be adjusted in height. They are easily folded up and down with no pinching risk for the user. If needed My-Loo can be used with only one armrest.



Simple, convenient installation

Etac My-Loo is installed quickly and easily using an innovative new solution. Practical, quick nuts make it possible to attach the raised toilet seat from above. The seat and armrests can then be clicked smoothly into their brackets.



Spacious and practical

Generous recesses and an extra-large seat opening provide excellent accessibility. The raised toilet seat's design, with free space around the seat, provides plenty of room for all body shapes.



Stable

My-Loo fixed sits securely in place and has a max user weight of 150 kg. For additional support and increased weight capacity, the accessory supporting legs can be added (max user weight 200 kg).



Safe and comfortable

With stable, easy-grip armrests and a well-designed seat, My-Loo makes toilet visits safe and comfortable for the user. For extra comfort, a soft seat can be added.



Easy to clean

The clean, smooth surfaces are easy to wipe with a soft cloth. Should separate cleaning be required, the seat and armrests are easy to remove from their brackets and equally simple to put back in place.



My-Loo with armrests
2 cm



My-Loo with armrests
6 cm



My-Loo with armrests
10 cm




My-Loo
6 cm



My-Loo
10 cm

Dimensions

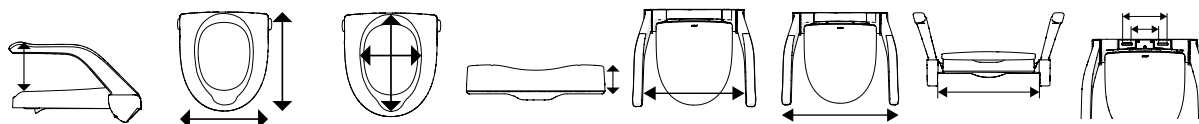

| Model | Height armrests | Width/Length | Inner dimensions | Height | Width between armrests | Total width | Attachment width (max toilet width) | Dimensions between holes |
|----------|-----------------|--------------|------------------|--------|------------------------|-------------|-------------------------------------|--------------------------|
| 80301510 | 215/245/285 mm | 360/400 mm | 215/285 mm | 20 mm | 480 mm | 570 mm | 400 mm | c/c 123-203 mm |
| 80301511 | 175/205/245 mm | 360/400 mm | 230/360 mm | 60 mm | 480 mm | 570 mm | 400 mm | c/c 123-203 mm |
| 80301512 | 135/165/205 mm | 360/400 mm | 230/360 mm | 100 mm | 480 mm | 570 mm | 400 mm | c/c 123-203 mm |
| 80301513 | - | 360/400 mm | 230/360 mm | 60 mm | - | - | 400 mm | c/c 123-203 mm |
| 80301514 | - | 360/400 mm | 230/360 mm | 100 mm | - | - | 400 mm | c/c 123-203 mm |

My-Loo 2 cm – functional toilet arm support

My-Loo 2 cm is the perfect arm support solution for around the toilet if you want to avoid drilling into the walls or floor. The product consists of armrests and a toilet seat mounted directly onto the toilet. As with the other versions with a raised seat, the armrests can be set to three different heights to suit the needs of different users. With its modern, discreet design, My-Loo blends in beautifully with most bathroom environments.



Etac My-Loo is delivered in a smart flatpack carrycase for easy handling

Accessories



Soft seat

80301604
Ergonomically shaped, soft and comfortable. Provides excellent pressure relief and comfortable support.
Material: Polyurethane



Top-mounted screw set

80301626
For toilets without through mounting holes (blind fitting). Should only be used with the included supporting legs. Suitable for mounting holes Ø20 mm. Material: Aluminium, polyamide, stainless steel, EPDM



Supporting legs

80301603
For additional stability. Increases max. user weight to 200 kg. Foldable.
Material: Aluminium, polyamide, polypropylene, TPE, stainless steel



Toilet paper holder

80301601
Attaches to the armrest.
Material: Polyamide, stainless steel



Storage basket

80301602
Attaches to the armrest.
Material: Polyamide, polyester/mesh, stainless steel



Cross bar

80303034
A secure support to hold on to. Attaches to the armrests. 60.5 cm
Material: Aluminium, polyethylene



Armrests

80301606
Sold in pairs, right, left version.
Material: Polypropylene





Etac is a world-leading developer and provider of ergonomic assistive devices and patient handling equipment. Our heart lies in the solutions that optimise quality of life for the individual, their family and caregivers.

For the latest news and continuously updated product information – visit: www.etac.com

Etac AB
Kista Science Tower
SE-164 51 Kista, Sweden
Tel +46 371 - 58 73 30
info@etac.se www.etac.com

 **etac**[®]
Creating Possibilities